

Select a New Level in Pain Blocking Power



MedFlo®



MedFlo® *MultiRate*

Pain Blocking Power Where You Need It



A Continuous Nerve Block Pump for Regional Anesthesia

Provide your patients unsurpassed pain control from start to finish with continuous regional anesthesia. Prolonging post operative analgesia improves patient comfort and allows rapid transition to an outpatient managed home care arrangement. Studies have shown that less than 2% of patients with continuous anesthesia requested any additional bolus for pain management.¹ Not only will your patients benefit from continuous regional anesthesia but so will your facility.



Advantages of Continuous Regional Anesthesia:¹

- Reduced nursing intervention, decreasing the chance of patient backlog.
- Improved patient well being including lower pain scores and increased ambulatory response.
- Facilitates same day discharge with home readiness within hours of surgery.
- Reduces the instances of post-surgical pain-related hospital readmissions.
- Results in increased operational efficiency which ultimately improves costs across several categories including nursing hours, patient volume and administration.

Ordering Information

Product Components:

The MedFlo MultiRate pump comes with a 1.2 micron filter, patient medication labels, 60 ml filling syringe and a protective carrying case.

MedFlo[®]MultiRate

Catalog #	Size	Flow Rates:
MR300	300ml reservoir	3, 4, 5 ml/hr, Prime rates
MR400	400ml reservoir	3, 4, 5 ml/hr, Prime rates
MR400H	400ml reservoir	6, 8, 10, 14 ml/hr
MR500H	500ml reservoir	6, 8, 10, 14 ml/hr
MR500S	500ml reservoir	3, 4, 5, 7 ml/hr
Additional MedFlo models available. Please contact your sales representative for more information. Questions about the MedFlo product? Call our 24-Hour MedFlo Support Line: 1-888-792-6127		

Indications:

1. The MedFlo MultiRate Pump is indicated for continuous delivery of local anesthetics in close proximity to nerves for post operative pain management.
2. The MedFlo MultiRate Pump is also indicated to significantly decrease narcotic use and pain when used to deliver local anesthetics in close proximity to nerves when compared with narcotic only pain management.
3. The MedFlo MultiRate Pump is disposable and single use only.

Contraindications:

1. Not for IV use. For regional block only.
2. Not for infusion of blood or blood products, lipids or fat emulsions or insulin.
3. Not for infusion of critical or life-supporting medications whose stoppage, interruptions, overdelivery or underdelivery would likely cause serious injury or death.
4. Not for use by patients who do not possess the mental, physical or emotional capability to self-administer their therapy; or who are not under the care of a responsible adult.

Warnings:

1. Do not disconnect your MedFlo MultiRate Pump early unless specifically told to do so by your healthcare provider.
2. Do not shower, swim or bathe with the pump.
3. Do not use if package or device is damaged. Product is sterile and non-pyrogenic in unopened, undamaged package.
4. Some studies have shown that under certain conditions, the continuous injection of local anesthetics into a joint space may cause serious injury. Any such use and associated risks should be discussed between patient and physician prior to treatment.

Acacia, Inc.
 web: www.acaciainc.com • email: info@acaciainc.com • phone: 800.486.6677
 Pat. No. 5,529,214 and other Patents Pending • ©2009 Acacia, Inc.
 1. Data on file at Acacia, Inc.

Acacia[™]
 Life Improving Infusion Solutions